

1



Your genetic test kit will arrive in a Courier Post envelope

On opening you will find a DHL transportation box that contains the genetic test kit.

2



Open this box, and inside is your genetic test kit

Don't discard this DHL box, as this has to be used for returning your completed sample for analysis

3



Inside the internal box is the ORAGENE DNA collection kit

Underneath the collection kit you will locate;

- How to provide a saliva sample
- Specimen bag
- Your Test Requisition Form

4



Collecting saliva sample

- Follow the instructions provided on the **"How to provide a saliva sample"**
- Or watch video:



5



Preparing saliva sample for transport to Invitae


On the tube we have placed a label with

- Your name
- NHI
- Date of Birth

On the label please complete the collection date: MM / DD / YYYY

Note: In step 6, you will be asked to add this date to the requisition form

6

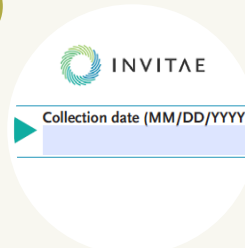

Complete the test requisition form

On the form write:
Collection date MM/DD/YYYY

Your details on the tube must exactly match those on the test requisition form

7



Place the tube in the specimen bag and seal.

To seal, peel the minigrip strip to expose the adhesive

Place the specimen bag along with the test requisition form into the internal box (collection kit box)

8



Place the collection kit box back in the DHL transportation box



Close the box - do not seal

The DHL courier driver is required to inspect the contents before sealing

9

See overleaf

9

Arrange collection of your kit

Please email us at ask@familycancer.co.nz with the following:

- Address of where the collection kit needs to be picked up from
- Location eg. Front door
- Special instructions

And we will take care of this for you.

(please don't call DHL directly, as they won't be able to assist without our account details)